

Dental Sterilization Solutions

By: **Avi Cohen** – Head of Medical Solutions, Objet Geometries Ltd.

Abstract

This whitepaper describes the different sterilization methods and technologies currently available in today's dental equipment sterilization market, with observation to Rapid Prototype models. Focus is given to the more common sterilization methods used or available for dental equipment in hospitals and clinics today. At the end of the paper, there is a conclusion and table, which summarizes the common sterilization methods. Six sterilization methods/products were isolated and ranked based on their sterile efficacy and applicability to medical instruments and Rapid prototype models.

Sterilization means the use of a physical or chemical procedure to destroy all microbial life, including highly resistant bacterial spores.

Bacterial spores are the most resistant of all living organisms because their capability to withstand destructive agents. Although the chemical or physical process used to destroy all pathogenic microorganisms including spores is not absolute, when all parameters of the sterilization process have been met, instruments, supplies and equipment are thought to be sterile.

An object should be sterilized depending on its intended use. Critical objects (those that enter sterile tissues or the vascular system or through which blood flows, such as implanted medical devices) require sterilization before use.

Items that touch mucous membranes or no intact skin, like endoscopes, respiratory therapy equipment, and diaphragms, require high-level disinfection, which would not be discussed in this whitepaper. Sterilization falls into the following three categories:

- High temperature/pressure sterilization

- Chemical sterilization
- Radiation sterilization

Common High Temperature Sterilization

A. Steam Autoclave

Steam autoclave is the oldest, safest, and most cost effective method of sterilization in the medical equipment industry. The steam reaches 121-148°C (250-300°F) in the pressure chamber at 15 P.S.I. The sterilization period is dependent on the temperature and size of load and can range from 10-60 minutes.

Configurable cycles allow the user to customize the sterilization cycle for items such as hard items, wrapped items, liquids in vented containers, waste, and glassware.

The fastest steam sterilization used is flash sterilization and it can be accomplished using either gravity-displacement, or pre-vacuum cycles. It is generally a high-speed steam sterilization of an unwrapped instrument or device for 3 to 10 minutes in 132°C saturated steam.

This type of sterilization is only intended for use in hospital operating rooms for urgently needed equipment.

In the steam autoclave process, microorganisms are killed by heat, and this is accelerated by the addition of moisture. Steam by itself is not sufficient for sterilization, and pressure that is greater than atmospheric is needed to increase the temperature of steam for thermal destruction of microbial life.

Solutions:

Steam autoclave is used mostly for surgical instruments. This method is not well suited for heat sensitive materials and instruments.

Many surgical instruments are not designed to withstand prolonged heat and moisture of the steam sterilization process. This leads to alternative sterilization categories: chemical sterilization and radiation sterilization, which allow heat and moisture sensitive materials to be sterilized.

Common Chemical Sterilization

A. Ethylene Oxide (EtO) Gas

Ethylene Oxide gas was introduced in the 1950's, and it is an effective, low temperature chemical sterilization method. It also takes longer than steam sterilization, typically, 16-18 hours for a complete cycle. Temperatures reached during sterilization are usually in the 50-60°C range.

Ethylene oxide (EtO) is a chemical agent that kills microorganisms, including spores. EtO gas must have direct contact with microorganisms on the items to be sterilized. Due to EtO being highly flammable and explosive in air, it must be used in an explosion-proof sterilizing chamber in a controlled environment.

Items sterilized by this process must be packaged with wraps and be aerated. The aeration time may be long and is needed to make sterilized items safe for handling and patient use.

In general, EtO gas is a reliable and safe agent for sterilization when handled properly.

Solutions:

EtO is used to sterilize items that are heat or moisture sensitive. Disadvantages of EtO gas are that it can leave toxic residues on sterilized items and it possesses several physical and health hazards to personnel and patients that merit special attention.

Since EtO poses several health hazards, there are currently two alternative technologies that are currently available: Sterrad, a plasma phase hydrogen peroxide-based sterilizing agent and Steris, a per acetic acid based technology.

B. Sterrad - Low Temperature Hydrogen Peroxide Plasma

Low temperature plasma sterilization was introduced to fill the gap between autoclave: high temperature steam

sterilization (safest, fastest and least expensive) and EtO gas sterilization, which leaves toxic residuals. It is a low temperature, non-toxic, but fairly expensive sterilization method. In this process, hydrogen peroxide is activated to create a reactive plasma or vapor.

The Sterrad system is a Hydrogen Peroxide Gas Plasma Sterilization system with an operating temperature range of 45-50°C. Operating cycle times range from 45-70 minutes, depending on size of system.

This sterilization system uses a combination of hydrogen peroxide and low temperature gas plasma to quickly sterilize most medical instruments and materials without leaving any toxic residues. Sterilization by this method occurs in a low moisture environment.

The Hydrogen Peroxide Plasma Process using Sterrad:

The process consists of two consecutive and equal sterilization phases.

Vacuum / Preplasma Stage:

- When a low pressure is achieved in the vacuum stage, low temperature air plasma is generated. This helps in removing residual moisture from the chamber. The system is then vented to atmospheric pressure at the end of this stage.

Sterilization Stage:

- Pressure in chamber is reduced and an aqueous solution of hydrogen peroxide is injected and vaporized into chamber.
- The hydrogen peroxide diffuses throughout the chamber, surrounds the items to be sterilized, and starts the inactivation of the microorganisms.
- After the pressure is reduced, applying radio frequency (RF) energy creates an electric field and thus forms low temperature plasma.
- Free radicals are generated in the plasma by breaking apart the hydrogen peroxide vapor. Once the activated components react with the organisms and kill them, they lose their high energy and re-combine to form oxygen, water vapor, and nontoxic by-products.
- This is half of the total sterilization process. The other half of the cycle is completed by repeating the above sterilization steps.
- At the completion of the second half cycle, the source of RF energy is turned off, vacuum is released, and chamber is returned back to atmospheric pressure by

introduction of filtered air.

Solutions:

This system is best suited to sterilize heat sensitive medical equipment such as endoscopic equipment. With two systems, Sterrad provides sterilization solutions for both smaller and larger Solutions.

C. Steris System | Sterile Processing System

The Steris System is another, more common, low temperature sterile processing system. It uses the Steris 20 Sterilant Concentrate that combines peracetic acid, a chemical biocidal agent, and a proprietary anti-corrosion formulation to kill microorganisms at low temperature. The process is achieved at a temperature of 50-56°C.

The Steris 20 Sterilant Concentrate is mixed with sterile water to create the solution that flows into the sterilization chamber. The sterilization time is 12 minutes.

This is followed by repetitive sterile water rinses to complete the process. The entire process is completed in less than 30 minutes for a standard cycle.

Peracetic acid, by itself, is an oxidant and disinfecting agent for liquid immersion.

It maintains its effectiveness when high levels of organic debris are present. It is an acetic acid plus an extra oxygen atom that reacts with most cellular components to destroy cells.

Solutions:

Only immersible instruments can be used with this method, and only a few instruments can be sterilized at one time. No packaging required.

D. Cidex OPA Solution – Alternative to Glutaraldehyde

Cidex® OPA Solution is a High Level Disinfectant (HLD) for use in reprocessing heat sensitive medical devices. Cidex OPA Solution provides high-level disinfection in 12 minutes at room temperature (20°C) and is particularly active against mycobacteria.

CIDEX OPA Solution has the broad materials compatibility of glutaraldehyde, requires no activation, and has minimal odor. The Cidex OPA

Solution is replacing Cidex and Cidex Plus due to their

toxicity concerns.

Solutions:

This solution is frequently used on surgical cameras (endoscopes). The item to be disinfected must be thoroughly cleaned and dried before immersion. After immersion, the item must be rinsed thoroughly with sterile water prior to use.

2.3. Common Radiation Sterilization

A. Gamma, Beta Sterilization

Irradiation is an effective sterilization method, but it is limited to commercial use only. The product to be sterilized is exposed to radiation for 10 to 20 hours, depending on the strength of the source. The highest temperatures reached in gamma sterilization are usually 30-40°C. Gamma radiation is popular for sterilizing before shipment and it can be done through the packaging.

The principal sources of ionizing radiation are beta particles and gamma rays.

Beta particles, free electrons, are transmitted through a high-voltage electron beam from a linear accelerator. These high-energy free electrons will penetrate into matter before being stopped by collisions with other atoms. This means their usefulness in sterilizing an object is limited by the density, thickness of the object

and by the energy of the electrons. These free electrons produce their effect by ionizing the atoms they hit, producing secondary electrons that kill microorganisms.

Cobalt 60 is a radioactive isotope capable of breaking down to produce gamma rays. Gamma rays are electromagnetic waves that have the ability to penetrate a much greater distance than beta rays before losing their energy from collision. Because they travel with the speed of light, they must pass through a thickness measuring several feet before making sufficient collisions to lose all of their energy.

Solutions:

The radiation can change the properties of some materials like plastics and have adverse effects on glues or adhesives.

As you can see, each of the six common methods discussed above have their advantages and disadvantages.

3. Other Sterilization Methods

In this section, the other, less common sterilization methods are described. These methods are either existing, being replaced by new sterilization, or are still in development.

3.1. Other High Temperature Sterilization

A. Dry Heat

This process is conducted at 160-170°C for a minimum of two hours. Due to its high temperatures, its applicable use is limited.

Dry heat in the form of hot air is used primarily to sterilize anhydrous oils, petroleum products, and bulk powders that steam and ethylene oxide gas cannot penetrate. In the absence of moisture, higher temperatures are required than when moisture is present because microorganisms are destroyed through a very slow process of heat absorption by conduction.

3.2. Other Chemical Sterilization

A. Glutaraldehyde (Cidex Plus and Cidex)

- Cidex Plus Solution is a disinfectant that is used to disinfect medical instruments.

It is a 3.4% alkaline glutaraldehyde solution, which has tuberculocidal and high-level disinfection capabilities. It achieves high-level disinfection in 20 minutes at 25°C and has up to a 28-day reuse life.

- Cidex Activated Dialdehyde Solution is used to disinfect medical instruments and endoscopes. This solution can also be used in an automated reprocessor. glutaraldehyde solution, which has tuberculocidal and high-level disinfection capabilities. It achieves high-level disinfection in 45 minutes at 25°C and has up to a 14-day reuse life.

Both have been used as a cold liquid high-level disinfectant for heat sensitive equipment.

Note: Cidex (glutaraldehyde) products are being withdrawn from the European market due to concerns that it is toxic and harmful to health care staff in hospitals. Also, the U.S. market is requiring glutaraldehyde-free chemical solutions, which led to the formulation of the Cidex OPA solution.

B. VHP MD Series

VHP (Vaporized Hydrogen Peroxide) MD Series Sterilization System for Medical Devices is yet another low temperature sterilization system. It is different from the Sterrad system, in that it only uses hydrogen peroxide in vaporized form for sterilization, while the Sterrad system uses

vaporized hydrogen peroxide to initiate the sterilization, and then plasma to complete the sterilization process.

In the VHP process, hydrogen peroxide vapor is injected into the chamber via a series of pulses to sterilize packaged medical and diagnostic devices. The cycle time is 2 hours and the operating temperature ranges from 30-40°C.

Note: This system is large capital equipment that uses hydrogen peroxide. On the other hand,

Steris Corporation's other product, the Steris System 1 is a portable tabletop system that

uses peracetic acid as the sterilant and it is used to sterilize equipment right before use.

C. Chlorine Dioxide

Chlorine Dioxide is a chemical liquid sterilization process. The best operating temperature range for this process is 25-30°C, while using low concentrations of ClO₂. The process requires 6 hours of contact time to achieve sterilization. The presence of organic matter reduces activity. A processor converts a compound of dilute chlorine gas with sodium chlorite to form ClO₂ gas and this gas is then exposed to the equipment in a sterilizing chamber.

D. Ozone

Ozone sterilizes by oxidation, a process that destroys organic and inorganic matter. It penetrates membrane of cells causing them to explode. In this process, a generator is used to convert oxygen to ozone, as a 6 to 12 percent concentration of ozone continuously flows through the chamber. Ozone penetration is controlled by vacuum pressure or by adding humidity. After the process is complete, oxygen is allowed to flow through the chamber to purge the ozone. The cycle time may be up to 60 minutes depending on the size of the chamber or load of items to be sterilized. Due to ozone gas being corrosive, and it being able to damage moisture sensitive equipment or models, there

has not been much use of it in the medical industry.

3.3. Other Radiation Sterilization

A. X-Ray Sterilization

This is a new developing process that is based on obtaining X-rays through conversion of electron beams. The X-rays produced have the same penetrating properties as the rays produced by Cobalt-60. But with this, treatment is faster, more flexible, and more environmentally friendly.

X-rays offer excellent product penetration in sterilization, thoroughly treating the surface and interior of a product.

Conclusion

There are many sterilization methods available on the market, and it is critical to know that different sterilization methods tailor to different material types. Many of the emerging sterilization methods are low temperature based. It is known that steam autoclave is the most widely used, inexpensive, and effective sterilization method that is currently available. Also, historically, many hospitals have relied on EtO- based sterilization systems, but due to environmental and safety concerns, they have been investigating alternatives. The white paper included also a discussion of the two common, low temperature sterilization alternatives to EtO Gas sterilization: Steris System 1 and Sterrad.

The best, most established, low temperature sterilization method that fills the gap between steam sterilization and EtO gas sterilization, is the hydrogen peroxide plasma technology developed by Sterrad. Although Sterrad is an expensive sterilization method, it is effective and versatile. Also, despite its longer cycle time, the VHP MD Series System has its benefits and will become a more common sterilization system in the future. Out of the common methods investigated, here are the relevant methods for medical equipment sterilization of Rapid Prototype models, which to be preferably using a low temperature process (i.e. Chemical and radiation methods):

Sterilization Categories		
	Common Methods	Others
<i>High Temp'/ Pressure</i>	<ul style="list-style-type: none"> • Steam Autoclave 	<ul style="list-style-type: none"> • Dry Autoclave
<i>Chemical</i>	<ul style="list-style-type: none"> • Ethylene Oxide (EtO) • Sterrad[®] • Steris System 1[®] • Cidex[®] OPA Solution 	<ul style="list-style-type: none"> • Gtutaraldehyde (Cidex Plus[®], Cidex[®]) • VHP[®] MD Series • Chlorine Dioxide • Ozone
<i>Radiation</i>	<ul style="list-style-type: none"> • Gamma 	<ul style="list-style-type: none"> • Ele' Beam (E-Beam)

ABOUT OBJET GEOMETRIES

Objet Geometries Ltd., the innovation leader in 3D printing for rapid prototyping and additive manufacturing, provides 3D printing systems that enable manufacturers and industrial designers to reduce cost of product development and dramatically shorten time-to-market of new products.

Objet's ultra-thin-layer, high-resolution 3D printing systems and materials utilize PolyJet[™] polymer jetting technology, to print ultra-thin 16-micron layers. The market-proven Eden[™] line of 3D Printing Systems and the Alaris[™]30 3D desktop printer are based on Objet's patented office-friendly PolyJet[™] Technology. The Connex[™] family is based on Objet's PolyJet Matrix[™] Technology, which jets multiple model materials simultaneously and creates composite Digital Materials[™] on the fly. All Objet systems use Objet's FullCure[®] materials to create accurate,

clean, smooth, and highly detailed 3D parts. All Objet systems use Objet's FullCure[®] materials to create accurate, clean, smooth, and highly detailed 3D parts.

Objet systems are in use by world leaders in many industries, such as Education, Medical / Medical Devices & Dental, Consumer Electronics, Automotive, toys, consumer goods, and footwear industries in North America, Europe, Asia, Australia, and Japan.

Founded in 1998, Objet serves its growing worldwide customer base through offices in USA, Mexico, Europe, Japan, China and Hong Kong, and a global network of distribution partners. Objet owns more than 50 patents and patent pending inventions. For more information, visit us at www.objet.com.

Objet Geometries Ltd.
Headquarters
2 Holzman st.,
Science Park,
P.O Box 2496,
Rehovot 76124, Israel
T: +972-8-931-4314
F: +972-8-931-4315

Objet Geometries Inc.
North America
5 Fortune Drive
Billerica,
MA 01821
USA
T: +1-877-489-9449
F: +1-866-676-1533

Objet Geometries GmbH
Europe
Airport Boulevard B 210
77836 Rheinmünster
Germany
T: +49-7229-7772-0
F: +49-7229-7772-990

Objet Geometries AP
Asia Pacific
Unit28, 10/f, HITEC
1 Trademart Drive
Kowloon Bay, Kowloon
Hong Kong
T: +852-217-40111
F: +852-217-40555

Objet Geometries AP
Limited China Rep Office
Rm1701, CIMIC Tower,
1090 Century Blvd,
Pudong Shanghai
200120 China
T: +86-21-5836-2468
F: +86-21-5836-2469

info@objet.com www.objet.com

© 2010 Objet, Quadra, QuadraTempo, PolyJet, FullCure, SHR, Eden, Eden250, Eden260, Eden260V, Eden330, Eden350, Eden350V, Eden500V, Job Manager, Objet Studio, CADMatrix, Connex, Connex350, Connex500, Alaris, Alaris30, PolyLog, TangoBlack, TangoBlackPlus, TangoGray, TangoPlus, VeroBlue, VeroWhite, VeroBlack, VeroGray, Durus, Digital Materials, PolyJet Matrix and ObjetGreen are trademarks of Objet Geometries Ltd. and may be registered in certain jurisdictions. All other trademarks belong to their respective owners.

